Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@brasseler.de
 www.brasseler.de

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Diamond discs with a diameter of > 30 mm

1. Indication

For separating saw models made of plaster.

2. Contraindication

There are no known contra-indications related to the instruments.

3. Warnings

- Maximum speed: \odot_{max} 15,000 rpm.
- Exceeding the maximum speed might lead to fracture of the instrument shank.
- Use full chucking length in order to minimize alternating bending loads which might lead to fracture of the instrument shank.
- Make sure that your hand is sufficiently supported during operation.
- If the handpiece falls down, please do not re-use the disc; with deformed discs there is the risk of jamming and fracture of the instrument.
- Prior to use, make sure that the instruments are not damaged.
- Important: If improperly used, disc may get stuck in the cutting groove which may lead to uncontrolled slipping of the disc. Increased risk

of injury!

4. Proper use

- Speed: \odot_{max} 10,000 rpm
- Make sure that the disc does not come into contact with the workpiece before the operating speed is reached
- Do not apply pressure, avoid jamming and leverage
- Clean clogged instruments with the cleaning stone
- Serrated diamond discs with teeth facing to the right, to be used in clockwise direction only

5. Safety and liability

Bent or out of true diamond discs must be eliminated and replaced by new ones immediately. Improper use can lead to damage to the power system or injury. The user is responsible to check the products prior to use as to whether they are suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to nonobservance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© Gebr. Brasseler GmbH & Co. KG