Komet Dental  
 Gebr. Brasseler GmbH & Co. KG  
 Trophagener Weg 25  
 32657 Lemgo - Germany  
 Telefon+49 (0)5261 701- 0  
 Telefax+49 (0)5261 701- 289  
 info@brasseler.de  
 www.brasseler.de

**Kombinationsinstrumente**  
 © Gebr. Brasseler GmbH & Co. KG  
 301123 - Rev 2019-02

DE EN FR ES IT

Füllungsbearbeitung, Work on fillings, Preparation des obturations, Trabajo en obturaciones, Lavorazione di otturazioni

Verpackungseinheit, Packing unit, Unité d'emballage, Unidad de embalaje, Unità d'imballo

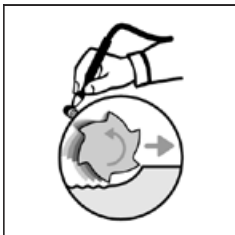
Fig. 1

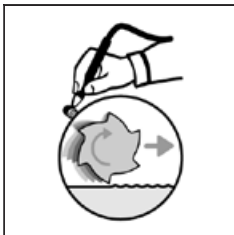
Fig. 2

DE

**Kombinationsinstrumente aus Hartmetall**

**1. Indikation**

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**Tungsten carbide combination instruments**

**1. Indication**

The instruments are intended for trimming and finishing plastic restoration materials.

**2. Contraindication**

There are no known contra-indications related to the instruments.

**3. Proper use**

- Use instruments after the cavity has been filled (adhesive restoration).
- Trimming and finishing is carried out step by step.
  - a) Trim the restoration surface with the selected instrument in the red contraangle at 20,000 rpm with usual coolant supply (50 ml/min.). The combination finishing instrument is used like a conventional finishing instrument, i.e. cutting to the right, counterclockwise (fig. 1: guiding the instrument antirotationally). Prepare the filling so that only a small amount of material has to be reduced when finishing.
  - b) Use the same instrument for final finishing of the filling unidirectionally by cutting to the left (fig. 2: guiding the instrument in direction of its rotation). Set the motor to anti-clockwise rotation. Recommended speed is 20,000 rpm in the red contra-angle with usual coolant supply (50 ml/min.).
- Apart from this general information, please observe the individual speed indicated on the packaging.
- The maximum permissible speed indicated on the package must not be exceeded in order to avoid risk of injury due to instrument breakage.

**4. Reprocessing**

The instruments are to be disinfected with anti-corrosive disinfection and cleaning agents for rotary instruments (e.g. with Komet DC1). For recommendations for use (immersion time, concentration, suitability) of disinfection and cleaning agents see instructions of the manufacturers of these agents. To remove disinfection and cleaning agent, thoroughly rinse instruments with water and dry carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instruments for a longer period in wet or humid condition. Make sure that they do

not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use, as amended, and more detailed information can be downloaded from our web site [www.kometdental.de](http://www.kometdental.de) or requested from the manufacturer Gebr. Brasseler.

**5. Storage**

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

**6. Disposal**

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

**7. Safety and liability**

Worn and damaged instruments (blunt blade, bent instruments etc.) have to be rejected and replaced by new ones. The above-mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

**8. Procedure in case of serious adverse events**

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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