Fig. 1

8934A.900.100	○ _{max} 30.000 min ⁻¹	○ _{min} 5.000 min ⁻¹
8934A.900.140	○ _{max} 30.000 min ⁻¹	○ _{min} 5.000 min ⁻¹
8934A.900.180	○ _{max} 25.000 min ⁻¹	○ _{min} 5.000 min ⁻¹
8934A.900.220	○ _{max} 20.000 min ⁻¹	○ _{min} 5.000 min ⁻¹

Fig. 2

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Diamond-coated stripping disc for proximal enamel reduction

1. Indication
 Interproximal reduction of enamel within the course of an orthodontic treatment.

2. Contraindication
 There are no known contra-indications related to the instruments.

3. Proper use
 - Make sure that the stripping disc is always used with a disc guard to avoid injury.
 - Place the disc occlusally or vestibularly and move it slowly through the contact area at a speed of 5,000 rpm.
 - Do not exceed the maximum speed (fig. 2).
 - Avoid contact with soft tissue in order to minimize risk of traumatization.
 - Make sure that sufficient coolant is applied.

4. Reprocessing
 The instrument is to be treated with anticorrosive disinfecting and cleaning agents for rotary instruments (e.g. Komet DC1). Regarding recommendations for use (immersion time, concentration, suitability) of the disinfecting or cleaning agents please refer to the instructions provided by the manufacturers. Disinfecting and cleaning agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instrument in wet or humid condition for a prolonged period of time. Make sure that instruments do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfecter, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Inspect cleaned instruments visually. Any damaged or blunt separating strips are to be rejected and discarded. Sterilization is carried out in the autoclave at 134°C. Please observe manufacturers' instructions of these devices. Corroded instruments must be rejected and their use discontinued. The operator of medical products is responsible for seeing that

proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use, as amended, and more detailed information can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

5. Storage
 Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

6. Disposal
 The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

7. Safety and liability
 Worn and damaged instruments (defective diamond coating, bent instruments etc.) have to be sorted out and replaced by new ones. The abovementioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. The user is responsible to check the product prior to use whether it is suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

8. Procedure in case of serious adverse events
 In case of serious adverse events, please proceed in line with the legal regulations in your country.

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