Komet Dental  
 Gebr. Brasseler GmbH & Co. KG  
 Trophagener Weg 25  
 32657 Lemgo · Germany  
 Telefon +49 (0) 5261 701-0  
 Telefax +49 (0) 5261 701-289  
 info@kometdental.de  
 www.kometdental.de

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Diamond-coated discs for finishing and polishing composite fillings

**1. Proper use**

1. After a composite filling has been placed, the finishing and polishing discs are used for the removal of excess material and step by step polishing.
2. Instruments have to be checked prior to each use. Do not use damaged discs.
3. All discs are to be mounted on Komet mandrel 310 exclusively – otherwise the integrated sliding clutch cannot be activated or the disc may detach. For safe mounting (dismounting) hold finishing disc between your middle finger and index finger and mount (dismount) applying light pressure on the mandrel with your thumb – see fig. 1.
4. Step by step procedure:
  - a. Removal of excessive material and finishing with the light blue polishing disc 9687. For surpluses with sharp edges use diamond finishing disc 952 because the polishing disc might get torn.
  - b. Pre-polishing with the pink polishing disc 9688.
  - c. High-shine polishing with the grey polishing disc 9689.

**2. The following instructions should be observed:**

- Recommended speed:  $\varnothing_{\text{rot}}$  5.000 – 6.000 rpm.
- Do not exceed the maximum permissible speed (see packaging).
- Use sufficient coolant (at least 50 ml/min.).
- Working at high speeds without coolant leads to excessive heat generation, smudging of composite and poor surface quality.
- Apply low contact pressure in order to minimize heat generation.
- Avoid axial deflection over 45° and radial deformation. This may lead to fracture of the disc.
- In order to guarantee a non-traumatic treatment, ensure direct vision and avoid contact with soft tissue.
- Use a rubberdam in order to avoid risk of aspiration.

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**Instructions for use**

Please read carefully and retain for future reference.

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- Applies to the diamond-coated finishing disc 952 only: An integrated sliding clutch stops the disc in case it gets jammed, in which case the finishing disc can no longer be used.

**3. Reprocessing**

Prepare the instruments (diamond-coated discs for finishing and polishing) before first use on the patient. Use disinfection agents which are suited for polishers (e.g. Komet DC1). Observe the manufacturer's recommendations regarding material compatibility, concentration and immersion time. Alcohol is not suited for disinfecting polishers. Reprocess polishers in the instrument bath because the vibrations in the ultrasonic bath might be absorbed by the elastic materials. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for polishers (mildly alkaline detergent). Inappropriate disinfection agents and improper handling might affect the characteristics of the polisher, the elastic bonding will become brittle and may break. Sterilise the instruments at 134°C in the autoclave only. Sterilisation in the chemiclave might lead to destruction. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use, as amended, and more detailed information can be downloaded from our web site [www.kometdental.de](http://www.kometdental.de) or requested from the manufacturer Gebr. Brasseler.

**4. Safety and liability**

Bent or out of true instruments must be eliminated and replaced by new ones immediately. Inappropriate use can damage the power system or cause injury. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© Gebr. Brasseler GmbH & Co. KG