Komet Dental
Gebr. Brasseler GmbH & Co. KG
Trophagener Weg 25
32657 Lemgo - Germany
Telefon+49 (0)5261 701-0
Telefax+49 (0)5261 701-289
info@brasseler.de
www.brasseler.de

Aluständer

© Gebr. Brasseler GmbH & Co. KG
301142 - Rev 2019-01

Verpackungseinheit, Packing unit, Unité d'emballage,
Unidad de embalaje, Unità d'imballò

DE

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1. Indikation

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Aluminium bur block

1. Indication

The Komet aluminium bur block is designed for individual composition of instruments for specific treatment methods and preparation techniques. This bur block assures an ergonomic, hygienic and economic procedure. The hinged lid prevents the instruments from falling out.

2. Contraindication

There are no known contra-indications related to the instruments.

3. Proper use

- Keep the bur block in its original package until first use.
- For convenient handling, the bur block can be equipped with the instruments needed for the individual treatments.

4. Reprocessing

Used and new instruments can be cleaned, disinfected and sterilized in the bur block. Appropriate use will avoid damage to the bur block.
- Disinfection and cleaning in the ultrasonic bath only in connection with a suitable disinfecting solution (e.g. Komet DC1).
- Observe manufacturer's instructions regarding material compatibility, concentration, immersion time.
- Remove any stains directly after disinfection.
- Important: Prior to sterilization, rinse the bur block under running water and dry thoroughly, especially in the perforations (e.g. by air-blasting).
- After rinsing and drying, sterilise the bur block in the autoclave at 2.2 bar and 134°C. If the bur block contains critical instruments

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Instructions for use

Please read carefully and retain for future reference.

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(invasive instruments) it has to be sealed in a sterilisation pouch.

Note: Multiple sterilizations of the bur block may lead to minor colour deviations. Such a change in colour does not have any influence on the quality or serviceability of the bur block. Due to its material, the aluminium bur block is not suitable for cleaning/disinfecting in the thermo disinfectant.

5. Storage

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

6. Disposal

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

7. Safety and liability

The user is responsible for checking the product prior to use as to whether it is suited for the intended purpose. The user is responsible for the proper use of the bur block. Damaged bur blocks are to be discarded immediately and replaced by new ones. Inappropriate use of the bur block can lead to damage to the instruments or injuries. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

8. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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