Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo · Germany
 Telefon +49 (0) 5261 701-0
 Telefax +49 (0) 5261 701-289
 info@kometdental.de
 www.kometdental.de

DF-Finierer
 © 02/2016, Gebr. Brasseler GmbH & Co. KG
 302248 - Rev 02/2016

Kronenpräparation, Crown preparation,
 Préparation coronaire, Preparación de coronas,
 Preparazione di corone

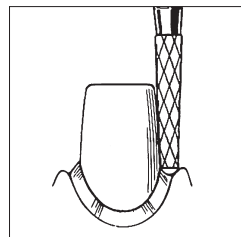
Abb./Fig. 1

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Tungsten carbide instruments with combined toothing for crown preparations

1. Indication

The instruments are designed for working on enamel and dentin. The combined toothing design produces a uniform, finely roughened crown wall and a smooth crown margin, guaranteeing secure retention and optimal marginal seal.

2. Proper use

- Those instruments that are supplied non-sterile have to be prepared prior to first use.
- The instruments are used after completion of the initial preparation (e.g. elimination of proximal contacts, determination of preparation depth, circular reduction of material) (Fig. 1).
- For shaping and finishing the corresponding speed ranges have to be observed.
- The initial preparation can be effected at a speed of up to \varnothing_{rot} 160.000 rpm.
- The subsequent finishing has to be effected at a speed of \varnothing_{rot} 20.000 - 30.000 rpm.
- Always use sufficient spray coolant (at least 50 ml/min).
- Apply only minimal contact pressure (< 2N).
- The maximum permissible speed indicated on the package must not be exceeded in order to avoid risk of injury due to instrument breakage.

3. Reprocessing

Those instruments that are supplied non-sterile have to be prepared prior to first use. The instruments are to be disinfected with anti-corrosive disinfection and cleaning agents for rotary instruments (e.g. with Komet® DC1®). For recommendations for use (immersion time, concentration, suitability) of

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Instructions for use

Please read carefully and retain for future reference.

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disinfection and cleaning agents see instructions of the manufacturers of these agents. Disinfecting and cleaning agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instruments for a longer period in wet or humid condition. Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664 can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

4. Safety and liability

Worn and damaged instruments (blunt blades, bent instruments etc.) have to be rejected and replaced by new ones. The above mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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