Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo · Germany
 Telefon +49 (0) 5261 701-0
 Telefax +49 (0) 5261 701-289
 info@kometdental.de
 www.kometdental.de

S-Dia Kronenstumpfpräparation
 © 08/2013, Gebr. Brasseler GmbH & Co. KG
302550 - Rev 08/2013

Kronenpräparation, Crown preparation,
 Préparation coronaire, Preparación de coronas,
 Preparazione di corone

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Diamond instruments with structured surface for primary crown preparations

1. Indication

The instruments are used for the initial crown preparation to reduce tooth substance around the tooth (after elimination of the proximal contact and determination of the preparation depth). Due to their structured surface the instruments ensure quick removal of tooth substance.

2. Contraindication

- The instruments are not designed for finishing or minimal reduction of tooth substance.
- They should not be used for working in cavities or on fillings.
- They are not suited for treatments where the instruments may be subject to jamming or leverage.

3. Clinical sequence/proper use

- The instruments are supplied non-sterile and have to be prepared prior to first use.
- After elimination of proximal contact and determination of the preparation depth, the instrument is guided around the tooth considering the high degree of material reduction. Shaping is carried out at a optimum speed of $n_{opt} = 160.000 \text{ rpm}$.
- Apart from this general information, please observe the individual speed indicated on the packaging.
- The maximum permissible speed indicated on the package must not be exceeded in order to avoid risk of injury due to instrument breakage.
- In order to guarantee a perfect preparation, the surface should be smoothed with a form congruent finishing instrument following the initial preparation.
- Always use sufficient spray coolant (at least 50 ml/min).
- Apply only minimal contact pressure (< 2N).

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Instructions for use

Please read carefully and retain for future reference.

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4. Cleaning, disinfection and sterilisation

The instruments are to be disinfected with anticorrosive disinfection and cleaning agents for rotary instruments (e.g. with Komet® DC1). For recommendations for use (immersion time, concentration, suitability) of disinfection and cleaning agents see instructions of the manufacturers of these agents. To remove disinfection and cleaning agent, thoroughly rinse instruments with water and dry carefully (e.g. by airblasting). Do not store instruments for a longer period in wet or humid condition. Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664 can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

5. Safety and liability

Worn and damaged instruments (defective diamond coating, bent instruments etc.) have to be sorted out and replaced by new ones. The abovementioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. The user is responsible to check the product prior to use whether it is suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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