Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@brasseler.de
 www.brasseler.de

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EN

Polishers interspersed with diamond grit for ceramics and ceramic veneers

1. Indication

Three to four-step polishing system for polishing ceramic materials and ceramic veneers. Depending on the quality of the surface to be polished, the first or the two first steps (blue, green) can be omitted.

2. Contraindication

There are no known contra-indications related to the instruments.

3. Proper use

- Make sure that only technically and hygienically perfect handpieces and contra-angles are used.
 - Collets have to be cleaned regularly.
 - Insert the polisher as deeply as possible.
 - Rotate polisher at optimum speed before contact is made.
 - Do not exceed the maximum permissible speed!
1. Step prepolishing: blue, green for rough surfaces and for shaping
 2. Step polishing: pink for regular surfaces

3. Step high-shine polishing: grey for fine surfaces

Note: Some shapes are also available in a super-coarse, green version for contouring. Before use, the polisher has to be examined for concentricity and safety. Only use the polisher for the intended application and always wear safety glasses. Apply low contact pressure (≤ 2 N) only. In order to achieve an even surface circular movements are recommended. The optimal speed is \varnothing_{max} 5,000 – 6,000 rpm. You will achieve best results when observing the recommended speed. Increased pressure and speed will lead to excessive wear and heat generation which finally might result in the destruction of the polisher. The maximum permissible speed is indicated on the label.
 Abrasive agent: Diamond grains

4. Safety and liability

Please observe the general instructions for use and safety recommendations in the current Komet dental catalogue. The user is responsible for checking each polisher prior to use to ensure that it is suited for the intended purpose. Bent or out-of-concentric instruments are to be discarded and replaced by new ones immediately. Improper use may lead to damage of the power system or injury. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to nonobservance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© Gebr. Brasseler GmbH & Co. KG