Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@brasseler.de
 www.brasseler.de

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Instructions for use

Please read carefully and retain for future reference.

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EN

Polishers for precious metal, non-precious metal alloys and model cast

1. Indication

The dark brown metal polisher (pre-polishing) is merely suitable for use on non-precious metal.
 The brown metal polisher (polishing) and the green metal polisher (high-shine polishing) are suitable for use on precious metal, non-precious
 metal and model cast alloys.
 Depending on the surface quality the pre-polishing step can be omitted.

2. Contraindication

There are no known contra-indications related to the instruments.

3. Proper use

- Only ever use properly maintained and cleaned power systems in good working order.
- Chucks must be cleaned in regular intervals.
- Polisher must be inserted to a sufficient depth.

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- Bring polisher up to optimal speed before application.

- Never exceed the maximum permissible speed!

Prior to polishing check the instrument's concentricity and ensure that it is safe to use. For your own safety, always wear protective glasses and
 only use polishers for the recommended application. Always apply low contact pressure (≤ 2 N). In order to achieve an even surface
 circular movements are recommended. The optimal speed is \varnothing_{max} 5,000 - 6,000 rpm. Best results are achieved if the indicated speed is
 observed. Excessive contact pressure and speed can lead to increased heat generation, which may result in the destruction of the polisher.
 The maximum permissible speed is indicated on the label.

4. Safety and liability

Bent or out of true polishers must be eliminated and replaced by new ones immediately. In case of contributory negligence by the
 user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our
 recommendations for use or warnings as well as inadvertent misuse by the user.

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© Gebr. Brasseler GmbH & Co. KG