Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@brasseler.de
 www.brasseler.de

Compositopolierer

© Gebr. Brasseler GmbH & Co. KG
 302588 - Rev 2019-03

Füllungsbearbeitung, Work on fillings, Preparation des obturations, Trabajo en obturaciones, Lavorazione di otturazioni

Verpackungseinheit, Packing unit, Unité d'emballage, Unidad de embalaje, Unità d'imballo

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Polishers interspersed with diamond grit for composite

1. Indication

3-step-system for polishing composite (micro, hybrid and macro), veneer acrylics and veneer materials filled with glass-ceramic. Depending on the quality of the surface to be polished, the first step can be omitted.

2. Contraindication

There are no known contra-indications related to the instruments.

3. Proper use

- make sure that only technically and hygienically perfect handpieces and contra-angles are used
- insert to full chucking length
- bring polisher to optimum speed before contact is made
- do not exceed the maximum permissible speed!

1. Step prepolishing: light blue, for coarse surfaces and for shaping
2. Step polishing: light pink or light green, for regular surfaces
3. Step high-shine polishing: light grey, for fine surfaces

Supply sufficient spray coolant in order to avoid smearing of the composite surface. Before use, the polisher has to be examined for concentricity and safety. Use the polisher only for the intended application and always wear safety glasses. Apply low contact pressure (≤ 2 N) only. In order to achieve an even surface circular movements are recommended. The optimal speed is \varnothing_{rot} 5,000 - 6,000 rpm. You will achieve best results when observing the recommended speed. Increased pressure and speed will cause excessive heat generation which finally might result in damage to the pulp and destruction of the polisher. The maximum permissible speed is indicated on the label. Abrasive material is diamond grit.

4. Reprocessing

Disinfect polisher before first use on the patient. Use disinfection agents which are suited for polishers (e.g. Komet DC1). Observe the

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Instructions for use

Please read carefully and retain for future reference.

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manufacturer's recommendations regarding material compatibility, concentration and immersion time. Alcohol is not suited for disinfecting polishers. Reprocess polishers in the instrument bath because the vibrations in the ultrasonic bath might be absorbed by the elastic materials. The instruments can be reprocessed in the thermo disinfecter, provided that the agent used is suitable for polishers (mildly alkaline detergent). Inappropriate disinfection agents and improper handling might affect the characteristics of the polisher, the elastic bonding will become brittle and may break. Sterilise the instruments at 134°C in the autoclave only. Sterilisation in the chemiclave might lead to destruction. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use and more detailed information, as amended, can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

5. Storage

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

6. Disposal

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

7. Safety and liability

Bent or out of true polishers must be eliminated and replaced by new ones immediately. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

8. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.