GEBR. BRASSELER GmbH & Co. KG

Trophagener Weg 25 · 32657 Lemgo

Postfach 160 · 32631 Lemgo · Germany

Telefon +49(0)5261 701-0

Telefax +49(0)5261 701-289

1103.302609.DRÖ6

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## GB Standard diamond instruments in small sizes

### 1. Indication

The instruments are designed for minimally invasive cavity preparation, i.e. for opening minimal carious defects and for intricate preparation

measures.

### 2. Precondition

- Preparations requiring high contact pressure (> 2N), quick reduction of tooth substance, removal of filling materials.
- Treatments where instruments might be subject to jamming and leverage.

### 3. Proper use

- We recommend to use the instruments with FG shank (314) at a speed of  $\varnothing_{opt.} 160\,000\text{ rpm}$  in the red contra-angle and the instruments with RA shank (204) at a speed of  $\varnothing_{opt.} 40\,000\text{ rpm}$ .
- Apart from this general information, please observe the individual speed indicated on the packaging.
- **Due to the small dimensions of the instruments only use minimal contact pressure (< 2N) in order to avoid that the instruments get dull quickly or break.**
- Always use sufficient spray coolant (at least 50 ml/min).
- Due to a better vision by using magnifying glasses it is possible to preserve a maximum of sound tooth substance.

### 4. Recommendations for maintenance

The instruments are to be disinfected with anti-corrosive disinfection and cleaning agents for rotary instruments (e.g. with KOMET DC1). For recommendations for use (immersion time, concentration, suitability) of disinfection and cleaning agents see instructions of the manufacturers of these agents. To remove disinfection and cleaning agent, thoroughly rinse instruments with water and dry carefully (e.g. by airblasting). Do not store instruments for a longer period in wet or humid condition. Make sure that they do not get in contact with each other during ultrasonic cleaning. Control cleaned instruments visually. Damaged and dull instruments must not be used any longer. Sterilization of the instruments is carried out using the

known methods in the autoclave (134°C), chemi-clave (126°C) or hot air sterilizer (180°C). The recommendations of the respective device manufacturers have to be observed. After sterilization, instruments have to be checked for corrosion on their surfaces. Corroded instruments must not be used any longer.

### 5. Safety and liability

Worn and damaged instruments (defective diamond coating, bent instruments etc.) have to be sorted out and replaced by new ones. The above-mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. The user is responsible to check the product prior to use whether it is suited for the intended purpose. GEBR. BRASSELER declines any liability if these safety instructions are not observed.

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© 2003, GEBR. BRASSELER GmbH & Co. KG