GEBR. BRASSELER GmbH & Co. KG
Trophagener Weg 25 · 32657 Lemgo
Postfach 160 · 32631 Lemgo · Germany
Telefon +49 (0) 5261 701-0
Telefax +49 (0) 5261 701-289

0908.303034.DR06 Führungsstiftinstrumente 0197

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GB Diamond instruments with guide pin

1. Indication

These instruments are used for circular reduction in crown preparations after elimination of the proximal contact and initial preparation of the tooth. Due to the fact that the guide pin is not diamond-coated, a defined preparation line

is created. Therefore, the instruments guarantee controlled and safe preparations.

2. Contraindication

Treatments where the instruments tend to jam or where they are used as a lever. In particular, proximal or interdental jamming of the guide-pin has to be avoided.

3. Proper use

- After the initial preparation of the tooth and dilatation of the sulcus with cord or retraction ring, the instruments with guide pin are moved around the tooth. Shaping is carried out at an optimal speed of $\varnothing_{opt.} 40\,000\text{ rpm}$. It is recommended to use new instruments for high material reduction.
- When medium grain or coarse grain instruments are used for shaping, the surface should be finished with a form congruent fine grain instrument at an optimal speed of $\varnothing_{opt.} 20\,000\text{ rpm}$.
- Apart from this general information, please observe the individual speed indicated on the packaging.
- Do not exceed the maximum permissible speeds. Otherwise there is a risk of instrument breakage.
- If necessary, the preparation line should be placed deeper with a form-congruent diamond instrument without guide pin.
- It is recommended to use magnifying glasses during the preparation.
- If possible, apply minimal contact pressure (< 2N).
- Always use sufficient spray coolant (at least 50 ml/min).

4. Cleaning, disinfection and sterilization

The instruments are to be disinfected with anti-corrosive disinfection and cleaning agents

for rotary instruments (e.g. with KOMET DC1). For recommendations for use (immersion time, concentration, suitability) of disinfection and cleaning agents see instructions of the manufacturers of these agents. To remove disinfection and cleaning agent, thoroughly rinse instruments with water and dry carefully (e.g. by airblasting). Do not store instruments for a longer period in wet or humid condition.

Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo-disinfector provided that the agent used is suitable for rotary instruments (e.g. with KOMET DCTherm, mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134° C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664 can be downloaded from our web site www.brasseler.de or requested from the manufacturer Gebr. Brasseler.

5. Safety and liability

Worn and damaged instruments (defective diamond coating, bent instruments, defective guide pin) are to be rejected and replaced by new ones. The above mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the

intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© 2008, GEBR. BRASSELER GmbH & Co. KG