Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo · Germany
 Telefon +49 (0) 5261 701-0
 Telefax +49 (0) 5261 701-289
 info@kometdental.de
 www.kometdental.de

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Diamond instruments with structured surface for cavity preparation

1. Indication

The instruments are designed for opening cavities and for pre-preparation. Thanks to their structured surface, the instruments allow speedy removal of tooth substance, thus ensuring a quicker preparation.

2. Contraindication

- The instruments are neither designed for finishing nor for a minimal reduction of tooth substance
- They are not suited for treatments during the course of which the instruments may be subjected to jamming or leverage

3. Proper use

- The instruments are supplied non-sterile and have to be prepared prior to first use.
- The carious lesion in the enamel is opened with the front part of the instrument, following which the cavity is precisely prepared to the desired shape using the structured diamond instrument. Please bear in mind that the instruments are capable of removing large amounts of material. The preparation is preferably carried out in the red contra-angle, at an optimal speed of \varnothing_{max} 120.000 rpm - 160.000 rpm; the instrument can also be used in the turbine, in which case it is absolutely essential to use sufficient coolant.
- Beside this general information, please observe the individual speed indicated on the packaging. Do not exceed the maximum permissible speed indicated on the packaging as this may lead to injury caused by fractured instruments.
- Subsequently, the cavity walls and the transitional zones are smoothed with a suitable finishing instrument
- Always use sufficient spray coolant (at least 50 ml/min.)
- Apply only minimal contact pressure (< 2N)

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Instructions for use

Please read carefully and retain for future reference.

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4. Cleaning, disinfection and sterilisation

The instruments are supplied non-sterile and have to be prepared prior to first use. The instruments are to be disinfected with anticorrosive disinfecting and cleaning agents for rotary instruments (e.g. with Komet® DC1®). For recommendations for use (immersion time, concentration, suitability) of disinfecting and cleaning agents see instructions of the manufacturer of these agents. To remove disinfecting and cleaning agent, thoroughly rinse instruments with water and dry carefully (e.g. by air blasting). Do not store instruments for a longer period in wet or damp condition. Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664 can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

5. Safety and liability

Worn and damaged instruments (defective diamond coating, bent instruments etc.) must be rejected and replaced by new ones. The above mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the hand piece or injury. The user is responsible for checking the product prior to use as to whether it is suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© 11/2013, Gebr. Brasseler GmbH & Co. KG