Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@brasseler.de
 www.brasseler.de

Keramikpolierer
 © Gebr. Brasseler GmbH & Co. KG
303885 - Rev 2020-08

Füllungsbearbeitung, Work on fillings, Preparation des obturations, Trabajo en obturaciones, Lavorazione di otturazioni

Verpackungseinheit, Packing unit, Unité d'emballage, Unidad de embalaje, Unità d'imballo

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Polishers for ceramics and ceramic veneers

- Indication**
2-step system for polishing ceramic materials and ceramic veneers.
- Contraindication**
There are no known contra-indications related to the instruments.

- Proper use**
 - Make sure that only technically and hygienically perfect handpieces and contra-angles are used.
 - Chuck the instruments as deeply as possible.
 - The instrument must be rotating at the optimum speed before contact is made with the tooth.
 - Do not exceed the maximum permissible speed!

1. Step polishing: grey for regular surfaces
 2. Step high-shine polishing: pink for fine surfaces
- Make sure to use sufficient spray cooling to prevent the formation of micro cracks. Prior to use on the patient, make sure that the instrument is safe to use and is running concentrically. For your own safety, the polishers must only be used for the intended purpose. Eye protection is recommended. Apply low contact pressure (≤ 2 N) during all steps of polishing. Polishing should always be carried out in circular movements to achieve even surfaces. Work at an optimal speed of $\text{O}_{\text{opt.}}$ 5,000 to 6,000 rpm. Do not exceed the recommended speed to achieve best polishing results. Exceeding the indicated contact pressure or speeds may lead to excessive heat generation, causing damage to the pulp, or to a destruction of the polisher. The maximum permissible speed is indicated on the label. Abrasive material is silicon carbide grit.

- Reprocessing**
Disinfect polisher before first use on the patient. Use disinfection agents which are suited for polishers (e. g. Komet DC1, CIDEZYME/Cidex OPA for a manual reprocessing procedure). Observe manufacturer's recommendations regarding material compatibility,

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Instructions for use
Please read carefully and retain for future reference.

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concentration and immersion time. Alcohol is not suited for disinfecting polishers. The polishers can be reprocessed in the thermo disinfecter, provided that the agent used is suitable for mechanical reprocessing of polishers (mildly alkaline detergent, e.g. Neodisher MediClean Forte). Inappropriate disinfection agents and improper handling might affect the characteristics of the polisher, the elastic bonding will become brittle and may break. Sterilize the instruments in the autoclave, verified in compliance with EN ISO 17665, at 134°C for at least 5 minutes. Sterilisation in the chemiclave or disinfection in the thermodisinfecter can lead to the destruction of the instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Information on proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use and more detailed information, as amended, can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

- Storage**
Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

- Disposal**
The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

- Safety and liability**
Bent or out of true polishers must be eliminated and replaced by new ones immediately. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

- Procedure in case of serious adverse events**
In case of serious adverse events, please proceed in line with the legal regulations in your country.

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