Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon +49 (0) 5261 701-0
 Telefax +49 (0) 5261 701-289
 info@kometdental.de
 www.kometdental.de

Compo System
 © 08/2012, Gebr. Brasseler GmbH & Co. KG
 303965 - Rev 08/2012

Füllungsbearbeitung, Working on fillings,
 Préparation des obturations, Trabajo en obturaciones,
 Lavorazione di otturazioni

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Compo System

1. Indications: CompoClips and CompoStrips are disposable. They are used for polishing composite, amalgam, glass ionomer cement, semi-precious metal and precious metal.

2. Contraindications:

- All treatments during the course of which the CompoClips might jam or bend unduly.
- Multiple use

3. CompoClips/CompoClips are thin, transparent foil disks with a honeycomb patterned coating of aluminium oxide particles on both sides. The three grit sizes are distinguished by the corresponding colour code (see system overview).

Recommendations for use:

- Use suitable Tungsten carbide finishers (such as Komet Q-finishers) or the black CompoClips (CC1SCV.900.090 or 130) for previous removal of excess composite and for retouching composite fillings.
- Only use CompoClips in combination with the Komet pop-on mandrel 310.204. The mandrel is suitable for multiple use.
- For safe mounting and removal, hold instrument between middle and index finger and place on the mandrel (for removal: disengage from mandrel), applying light pressure. A noticeable click confirms safe locking.
- Bring the CompoClip to the required optimal speed of \odot_{max} 10.000 rpm prior to application to the tooth. The maximum permissible speed is \odot_{max} 16.000 rpm.
- Always provide sufficient cooling (at least 50 ml/min.); additional external cooling aimed at the operational site is required when working with the front end.
- Observe the specified sequence (blue – red – white)
- Rinse all residues off the surface.
- Avoid high contact pressure.
- Avoid contact with soft tissue.
- The operating time per CompoClip is 5 – 20 sec., according to the respective application.
- Improper use, such as excessive speed or pressure, can lead to overheating and damage to the surface to be restored as well as the destruction of the CompoClip. Risk of injury!

4. CompoStrips/CompoStrips are foil strips coated with aluminium oxide particles on one side. The three grit sizes are applied in a honeycomb pattern and arranged in succession on one strip. They can be distinguished by their different colours. Thanks to their transparency and their thin, flexible structure as well as their honeycomb

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coating and their two uncoated areas, the CompoStrips can be employed safely and successfully even in proximal zones with narrow contact points as well as close to the gingiva.

Recommendations for use:

- The uncoated surfaces serve to insert the strips, whereas the three blue stripes at the ends provide grips to facilitate handling.
- The narrow CompoStrip (CS20) is to be threaded from vestibular direction, below the contact point.
- If the entire proximal surface is to be worked on, the wider CompoStrip (CS40) should be used. This strip is inserted from coronal direction.
- Observe the correct sequence, i.e. blue – red – white.

5. Recommendations for maintenance The mandrels are to be treated with anticorrosive disinfecting and cleaning agents for rotary instruments (e.g. Komet DC1). Regarding recommendations for use (immersion time, concentration, suitability) of the disinfecting or cleaning agents please refer to the instructions provided by the respective manufacturers of these agents. Thoroughly remove all residues of disinfecting and cleaning agent with water and allow to dry completely (e.g. by air stream). Do not leave or store mandrels in wet or damp condition for a prolonged period of time. Inspect cleaned mandrels visually. Any damaged mandrels are to be rejected and discarded. Only the mandrels can be sterilized, contrary to the disks and strips which are disposable and therefore cannot be sterilized. Sterilisation of the mandrels is carried out in the autoclave at 134°C. Observe instructions provided by the manufacturer. After completed sterilization check surfaces for corrosion. Discard any corroded mandrels.

6. Safety and liability The disks and strips are disposable and have to be discarded directly after use. The reuse of the product(s) carries a risk of infection. A safe, risk-free use can therefore not be guaranteed. The Komet mandrels are suitable for multiple use. Worn and damaged mandrels are to be discarded and immediately replaced by new ones. The above instructions regarding handling and cooling are to be strictly observed. Only use the disks, strips and mandrels according to their intended purpose. The non-observance of these safety instructions can lead to failure of the product. It is the responsibility of the user to check prior to use that the product is suitable for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

Pop-on mandrel: German patent DE 197 11 912

CompoStrip: German utility patent GM 20 2004 019 490