Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@brasseler.de
 www.brasseler.de

ZR-Schleifer FT
 © Gebr. Brasseler GmbH & Co. KG
304504 - Rev 2019-12

Verpackungseinheit, Packing unit, Unité d'emballage,
 Unidad de embalaje, Unità d'imballo

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Instructions for use

Please read carefully and retain for future reference.

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Abrasive diamond instruments for working on ceramic primary crowns

1. Indication

Four stage system of abrasive instruments for working on all-ceramic materials according to the double crown technique by means of a water-cooled precision laboratory turbine inserted in a milling device. In order to guarantee that the instruments maintain their shape and their homogenous surface, do not use these abrasives freehand but only in a milling device.

2. Contraindication

There are no known contra-indications related to the instruments.

3. Proper use

Shaping of the surface of primary crowns in four stages. In order to achieve optimal results, the abrasive instruments should be used in "coarse to fine" order, as follows:

1. Grit size M (see REF no.) with blue/white colour coding and golden shank
2. Grit size F (see REF no.) with red/white colour coding and golden shank
3. Grit size EF (see REF no.) with yellow/white colour coding and golden shank
4. Grit size UF (see REF no.) with white/white colour coding and golden shank

The first stage serves to create the outer shape of the primary crown. During the following steps, the different stages of the finishing process, i.e. fine and ultra-fine grinding and high shine polishing, are carried out. \odot_{min} 160,000 rpm / \odot_{max} 300,000 rpm.

Note:

These abrasive instruments may only be used in laboratory turbines with irrigation facility which must be coupled to a milling device.

Do not exceed a contact pressure of < 2 N to guarantee effective processing of the ceramic material. Avoid jamming and seizing of the instruments. Do not use as a lever. In order to warrant a smooth operation without deflecting, make sure that the abrasive instrument is firmly locked in the final position of the chucking mechanism at all times!

Attention!

When working on ceramics with abrasive diamond instruments in the turbine, always make sure that sufficient irrigation is provided in order to reduce the risk of micro fissures.

4. Maintenance

The abrasive instruments are to be cleaned with a brush and in the ultrasonic bath. Do not leave instruments in the ultrasonic device for a prolonged period of time as this may damage the diamond coating. Acids and chemical vapours might cause corrosion on the instruments.

5. Safety and liability

Worn and damaged abrasives (i.e. damage to the diamond coating, bent instruments etc.) are to be discarded and immediately replaced by new ones. The above instructions regarding operation, cooling and contact pressure are to be strictly observed. Do not exceed the maximum speed indicated on the packing. Risk of injury due to fracture of the instrument! Only use the abrasive instruments according to their intended purpose. The disregard of these safety instructions can lead to damage to the power system or injury. It is the responsibility of the user to check prior to use that the product is suitable for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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