Komet Dental  
 Gebr. Brasseler GmbH & Co. KG  
 Trophagener Weg 25  
 32657 Lemgo · Germany  
 Telefon +49 (0) 5261 701-0  
 Telefax +49 (0) 5261 701-289  
 info@kometdental.de  
 www.kometdental.de

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**Dressing block for polishers**

**1. Product description**

Dressing of abrasives with ceramic bond, elastic polishers and separating discs with resin bond. The product consists of a dressing block and a mountable handle, if required (fig. 1).

**2. Start-up and appropriate use**

**Stationary use:**

The dressing block (without handle) can be mounted to the work station, which means that the dressing process can take place without the user having to put down the work piece. The dressing block is placed in front of the suction device and firmly mounted to the work station, ideally with two-component glue or by means of a screw connection (thread M4, fig. 2).

**Hand-held use**

Two threaded holes, one at the side and the other underneath the dressing block, allow optimum assembly of the handle if hand-held use is required. Alternatively, the handle can also be mounted elsewhere (fig. 3). Attention: When mounting the handle, make sure that it is screwed in tightly enough! For added safety and to prevent the handle from accidentally coming off, use glue or a special screw securing adhesive.

**Proper handling**

1. Insert the tool into the hand-piece as usual and dress at a speed of 5,000 – 15,000 rpm. Attention: Do not exceed the maximum speed of the tool.
2. Keep the contact pressure as low as possible. Do not work at the same spot for too long to avoid excessive wear of the dressing block.
3. The main part of the dressing procedure should be done on the surfaces of the dressing block. The V-shaped flute is slightly arched, which is why it is particularly suitable for correcting the shape of lenticular polishers, whereas the semicircular groove is suited for rounding polishing wheels.
4. During the dressing procedure, make sure that all dust is sucked off and protect your eyes.

**3. Safety and responsibility**

The user is responsible for checking the products prior to use to ensure that these are suited for the intended purpose. The user is responsible for the correct use of this dressing block. Inappropriate use of the dressing block can lead to injury. In case of contributory negligence by the user, Gebr. Brasseler partly or totally declines liability for all resulting damages, particularly due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© 08/2012, Gebr. Brasseler GmbH & Co. KG