Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@brasseler.de
 www.brasseler.de

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PolyBur – Polymer instrument for excavation

1. Indication

- The PolyBur is used for gentle caries excavation in the vicinity of the pulp of clinically symptom free milk and permanent teeth.
- Removal of soft, carious dentin mainly in milk teeth and „hidden caries“ in permanent teeth. In cases of „hidden caries“, the tooth looks perfectly intact from the outside without any visible damage to the enamel. There are, however, carious lesions in the deeper regions of the tooth (dentin) which can be detected with special diagnostic methods (i.e. X-ray, laser examination).

2. Contraindication

- Caries that has already caused clinical symptoms.
 - Treatments where high contact pressure is required
 - Dark, discoloured dentin (e.g. Maillard reaction)
 - Hard, demineralised dentin
 - Caries along the dentino-enamel junction
- In the above cases, the PolyBur is unable to remove an appreciable amount of substance and a conventional therapy is indicated.

3. Proper use

The instrument is used at a speed of $\varnothing_{rot.} 2.000 \text{ rpm}$ ($\varnothing_{max.} 8.000 \text{ rpm}$). Cooling with water spray is possible but not essential. The access to the cavity is created in the usual manner with rotary or oscillating instruments (diamond abrasives or tungsten carbide). Carious dentin is removed from the periphery of the cavity by means of conventional round burs. The PolyBur is used in the softer central areas. Soft defects should be removed first to preserve the cutting force of the instrument for as long as possible. Always treat the entire surface first, achieving an even amount of pressure, before minimally increasing the contact pressure and moving on to deeper layers. Due to the slim neck and the characteristics of the material, make sure to apply low pressure only (< 2 N) to avoid instrument fracture. The use of visual aids is recommended.

A decisive factor for the success of treatments to maintain the vitality of the tooth is a permanent, absolutely bacteria-proof restoration. This is of essential importance in selective caries removal, because this cuts the supply of nutrients to the remaining bacteria, thereby preventing the residual carious lesion from spreading. Adhesive restorations meet these requirements, provided that they have been implemented correctly.

Special notes:
 If the pulp is opened during the treatment, the damage is too advanced and an endodontic therapy is required.

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Instructions for use

Please read carefully and retain for future reference.

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After the use of the PolyBur, the surface of the dentin is not as hard as usual due to the partial demineralization of the dentin. This means that:

- The normal control with a probe cannot be carried out. The dentin is not going to clink and the probe might leave scratches on the surface of the dentin.
- The finished filling might look like a recurrent carious lesion on the radiograph due to the reduced absorption of x-rays. This proves that the PolyBur works and that any dentin worth saving is preserved. The dentin is going to change over the next few months, in an ideal scenario, it will remineralize.

4. Reprocessing

The PolyBur is disposable and ready to use, which means it does not need to be sterilised prior to use. The packaging is not suitable for sterilisation. If this is still required, the instrument removed from the pack can be sterilised in the autoclave at 134°C without causing damage to the material.

5. Storage

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

6. Disposal

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

7. Safety and liability

The PolyBur is designed for single use. Discard used instruments immediately. The above instructions regarding handling, cooling and contact pressure are to be strictly observed. Only use the instruments according to their intended purpose. The non-observance of these safety instructions can lead to damage to the hand piece or failure of the treatment. It is the responsibility of the user to check the products prior to use as to whether they are suited for the intended purpose. The user is responsible for the correct use of the instruments. In case of contributory negligence by the user, Gebr. Brasseler partly or totally declines liability for all resulting damages, particularly due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

8. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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© Gebr. Brasseler GmbH & Co. KG