Komet Dental
Gebr. Brasseler GmbH & Co. KG
Trophagener Weg 25
32657 Lemgo - Germany
Telefon +49 (0) 5261 701-0
Telefax +49 (0) 5261 701-289
info@kometdental.de
www.kometdental.de

EasySeal
© Gebr. Brasseler GmbH & Co. KG
308123 - Rev 2017-11

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EN
EasySeal – Root filling material on the basis of epoxy resin

1. Indication
EasySeal is a permanent root filling material based on epoxy-amine chemistry, used for orthograde filling techniques. EasySeal is radiopaque, dimensionally stable and free from iodoform and paraformaldehyde. The tissue will not react with the material.
Components:

Paste 1: 4-[2-(4-hydroxyphenyl)propan-2-yl]phenol-Epichlorhydrinresin, Alkylglycidyl ether, Barium sulfate, tri-calcium phosphate, diphenylolpropan-diglycidyl ether
Paste 2: Polyalkoxyalkylamine-copolymer, 5-amino-1,3,3-trimethylcyclohexanmethylamin, Aqua, Barium sulfate, tri-calciumphosphate, nanodispers silicon dioxide, polyhexamethylene biguanides-hydrochloride

The material fulfils the requirements of the ISO standard 6876 dental root canal sealing materials.

2. Contraindication
Do not use the material on patients who are allergic to epoxy resins, amines or other components of the product.

3. Proper use
Preparation of EasySeal:
Before use, remove the cap of the double-chamber syringe by turning counterclockwise. Check the level of the two components in the double-chamber syringe (see Fig. 1). Attach a mixing cannula/mixing tip (and Endo tip) supplied with the product by inserting the spring of the mixing cannula into the groove in the syringe. Tighten the cannula by turning in a clockwise direction. By pressing the piston, both pastes are mixed in the mixing cannula.
Application:
Prior to use EasySeal, the root canal has to be cleaned and subsequently dried by means of paper points and oil-free air jet during approx. 10 seconds. Fill the root canal with EasySeal:
- The paste can be applied and inserted from a mixing block, by means of a lentulo in rotary movements
- Select the corresponding Masterpoint, cut at the apical end, apply sealer and insert into the canal

Working time/Setting time:
The mixed root filling paste is workable for approx. 15 minutes. The use of thermal techniques will reduce the workable time. Full setting is reached at 37°C body temperature within 24 hours. After use it is absolutely necessary to remove the mixing cannula and close the product with the lid. Otherwise premixed material in the cannula may flow back into the syringe, causing inadvertent curing of the pastes in the double chamber syringe. Store at room temperature (20 - 25°C).

Important notes!
Do not allow unpolymerised pastes to come in contact with the skin or eyes. If eye contact occurs, rinse thoroughly with water immediately. Avoid contact with mucous membranes. If the root filling material comes in contact with mucous membranes remove it with suitable materials (e.g. cotton pellets). Rinse the oral mucosa with plenty of water. In the event of continuing sensitization, seek medical advice. If accidentally swallowed, drink plenty of water. In the event of nausea or feeling unwell, seek medical advice.

4. Further treatment
Fill the root canal by means of condensation or thermoplastic techniques using gutta-percha. Cleaning of instruments: Clean all instruments and mixing slabs immediately after use with a 50% solution of ethanol in water and detergent.

5. Storage
Store EasySeal at room temperature and in a dry place (do not refrigerate). Do not expose to direct sunlight. Once opened the double chamber syringe, use the paste within 6 months. Do not use after the expiry date. Keep out of the reach of children. Improper storage will reduce the shelf life and may impair the product's effectiveness.

6. Safety and liability
The user is responsible for checking the products prior to use as to whether they are suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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