Komet Dental  
Gebr. Brasseler GmbH & Co. KG  
Trophagener Weg 25  
32657 Lemgo · Germany  
Telefon +49 (0) 5261 701-0  
Telefax +49 (0) 5261 701-289  
info@kometdental.de  
www.kometdental.de

**Spüladapter SF1976 für Kupplung 9969**  
© 08/2012, Gebr. Brasseler GmbH & Co. KG  
308394 - Rev 08/2012

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**EN Rinse adapter for the reprocessing of the coupling 9969 in a washer/disinfector**

**1. Indikation**

The reusable rinse adapter SF1976 is specially developed for the validated cleaning and disinfection of the coupling 9969 made by KOMET in the washer/disinfector.

**2. Proper use**

The validated reprocessing procedure cannot be applied to couplings produced by other manufacturers. Place the mandrel (MULTiflex®) of the coupling 9969 in the rinse adapter SF1976 (fig. 1). Then screw the external thread of the KOMET rinse adapter in the silicon holders on the injector bar (fig. 2 - 4). Warning: The coupling must always be securely attached to the rinse adapter. The rinse adapter must always be firmly attached to the injector bar of the washer/disinfector. For a correct reprocessing result, it is absolutely essential that the above instructions be followed.

**3. Accessories**

9988 O-ring  
The black O-ring inside the rinse adapter SF1976 is subject to normal wear, which is why it has to be exchanged regularly after 12 months or earlier if the O-ring starts to wear out. To do this, remove the old O-ring from the rinse adapter with the help of a pointed probe. Insert the new O-ring in the rinse adapter with non-cutting tweezers/probe and press firmly into the inner groove provided.

**4. Safety and liability**

The user is responsible for checking the product prior to use as to whether it is suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partly or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

\*MULTiflex® is a registered trademark of the co. KaVo Dental GmbH, Biberach, Germany.

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\*MULTiflex est une marque déposée de la société KaVo Dental GmbH, Biberach, Allemagne
\*MULTiflex® es una marca registrada de la empresa KaVo Dental GmbH Biberach
\*MULTiflex® è un marchio registrato della ditta KaVo Dental GmbH Biberach.