Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo - Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@kometdental.de
 www.kometdental.de

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Diamond instruments with structured surface for cavity preparation

1. Indication

The instruments are designed for opening cavities and for pre-preparation. Thanks to their structured surface, the instruments allow speedy removal of tooth substance, thus ensuring a quicker preparation.

2. Contraindication

There are no known contra-indications related to the instruments.

3. Proper use

- The instruments are supplied non-sterile and have to be prepared prior to first use.
- The carious lesion in the enamel is opened with the front part of the instrument, following which the cavity is precisely prepared to the desired shape using the structured diamond instrument. Please bear in mind that the instruments are capable of removing large amounts of material. The preparation is preferably carried out in the red contra-angle, at an optimal speed of O_{rot} 120,000 rpm - 160,000 rpm; the instrument can also be used in the turbine, in which case it is absolutely essential to use sufficient coolant.
- Beside this general information, please observe the individual speed indicated on the packaging. Do not exceed the maximum permissible speed indicated on the packaging as this may lead to injury caused by fractured instruments.
- Subsequently, the cavity walls and the transitional zones are smoothed with a suitable finishing instrument.
- Always use sufficient spray coolant (at least 50 ml/min.).
- Apply only minimal contact pressure (< 2 N).

4. Reprocessing

Those instruments that are supplied non-sterile have to be prepared prior to first use. The instruments are to be disinfected with anti-corrosive cleaning and disinfection agents for rotary instruments (e.g. with Komet DC1). For recommendations for use (immersion time, concentration, suitability) of cleaning and disinfection agents see instructions of the manufacturers of these agents. Cleaning and disinfection agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instruments for a longer period in wet or humid condition. Make sure that

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Instructions for use

Please read carefully and retain for future reference.

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they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134°C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Information on proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use and more detailed information, as amended, can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

5. Storage

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

6. Disposal

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

7. Safety and liability

Worn and damaged instruments (defective diamond coating, bent instruments etc.) have to be rejected and replaced by new ones. The above mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. The user is responsible to check the product prior to use whether it is suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

8. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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© Gebr. Brasseler GmbH & Co. KG