Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo · Germany
 Telefon +49 (0) 5261 701-0
 Telefax +49 (0) 5261 701-289
 info@kometdental.de
 www.kometdental.de

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Diamond instruments for micro-preparation

1. Indication

The instruments are used for minimally invasive cavity preparation. The small sizes (889M, 838M and 830RM) are designed for fissure caries as well as opening of larger, deep carious lesions (Fig. 1). Furthermore, they can be used for difficult and specific preparations, e.g. bevelling of cavity margins in the proximal area or excavation in deep cavities near the pulp. In case of undermining caries (Fig. 2), the cavity can be prepared in the shape of an ampoule with the larger instruments (830M, 953M and 953AM). The elliptic shapes of figures 953AM, 953M and their diamond-coated neck permit the preparation of rounded surfaces with undercuts, especially for inner surfaces in the proximal area (Fig. 3). With adhesive restorations, the cavity margins should be rounded in order to absorb the strain between restoration and tooth substance, using the ballshape (830M) (Fig. 4). Instruments with medium grit can also be used for excavating in the red contra-angle at low speed and with sufficient air cooling. Apply minimal contact pressure and work in intermittend mode.

2. Contraindication

- Crown preparation, quick removal of tooth substance, drilling out and removal of filling materials.
- Treatments where the instruments can jam or where they are used as lever.

3. Proper use of the instruments

- The instruments are supplied non-sterile and have to be prepared prior to first use.
- The instruments are preferably used in the red contra-angle at the speed indicated in tab. 1 and with sufficient water cooling (at least 50 ml/min).
- Due to the long and thin neck, the instruments should be used with light contact pressure (< 2N) in order to avoid fracture.
- The use of visual aids will largely contribute to the conservation of sound tooth substance.
- Medium and fine grit instruments can be chosen depending on the material reduction or surface roughness required.

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4. Cleaning, disinfection and sterilization

The instruments are supplied non-sterile and have to be prepared prior to first use. The instruments are to be disinfected with anti-corrosive disinfection and cleaning agents for rotary instruments (e.g. with Komet® DC1[®]). For recommendations for use (immersion time, concentration, suitability) of disinfection and cleaning agents see instructions of the manufacturers of these agents. Disinfecting and cleaning agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instruments for a longer period in wet or humid condition. Make sure that they do not come in contact with each other during ultrasonic cleaning. The instruments can be reprocessed in the thermo disinfector, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued. Sterilization is carried out in the autoclave at 134 °C. The recommendations provided by the manufacturer of the respective device must be observed. Discard any corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664 can be downloaded from our web site www.kometdental.de or requested from the manufacturer Gebr. Brasseler.

5. Safety and liability

Worn and damaged instruments (defective diamond coating, bent instruments etc.) have to be discarded and replaced by new ones. The above mentioned recommendations with respect to handling, cooling and contact pressure are to be strictly observed. The instruments should only be used for the intended application. Non-observance of these safety recommendations may lead to damage of the handpiece or injury. The user is responsible to check the product prior to use whether it is suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

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© 06/2014, Gebr. Brasseler GmbH & Co. KG