 Komet Dental
 Gebr. Brasseler GmbH & Co. KG
 Trophagener Weg 25
 32657 Lemgo · Germany
 Telefon+49 (0)5261 701- 0
 Telefax+49 (0)5261 701- 289
 info@brasseler.de
 www.brasseler.de

Guttapercha Remover
 © Gebr. Brasseler GmbH & Co. KG
310583 - Rev 2018-11

DE EN FR ES IT

 Wurzelkanalaufbereitung, Root canal preparation, Traitement endodontique, Preparación de canal radicular, Strumentazione canolare

Verpackungseinheit, Packing unit, Unité d'emballage, Unidad de embalaje, Unità d'imballo

strahlensterilisiert (Sterile Instrumente sind auf dem Verpackungsetikett entsprechend gekennzeichnet), Sterilized using irradiation (Sterile instrumnts are marked as such on the label of the packaging), Stérilisation par irradiation (Les instruments stériles sont marqués comme tels sur l'étiquette de l'emballage), Esterilización por radiación (Los instrumentos estériles son marcados como tales sobre la etiqueta del embalaje), Sterilizazione per mezzo di radiazione (Gli strumenti sterili sono marcati come tali sull'etichetta dell'imballaggio)

verwendbar bis (nur zutreffend bei sterilen Instrumenten), Use-by date (only applies to sterile instruments), A utiliser avant le (ne s'applique qu'aux instruments stériles), Utilizar antes de (se aplica solamente a instrumentos estériles), Data di scadenza (riguarda soltanto strumenti sterili)

bei beschädigter Verpackung nicht verwenden (nur zutreffend bei sterilen Instrumenten), Do not use in case of damaged packaging (only applies to sterile instruments), Ne pas utiliser si l'emballage est endommagé (ne s'applique qu'aux instruments stériles), Non utilizar si el embalaje está dañado (se aplica solamente a instrumentos estériles), Non utilizzare se l'imballo è danneggiato (riguarda soltanto strumenti sterili)

DE

Guttapercha Remover

1. Indikation

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Guttapercha Remover

1. Indication

The instruments are intended for the mechanical removal of diseased tissue and root filling materials within the course of a root canal treatment.

2. Contraindication

The contra-indications are (e. g.):

- pronounced abrupt curves
- severely calcified root canals
- preparation of milk teeth
- revisions of carrier based filling systems with metal core

3. Proper use

Those instruments that are supplied non-sterile have to be prepared prior to first use.

The guttapercha remover is used for the revision of root canal fillings after the exposure of the canal access cavity. The instrument is provided with a safe end and "blunt" toothing which generates friction heat during use. The guttapercha remover is used without cooling, in rotary mode at a speed of ○_{rev.} 2,000 rpm in the blue contra-angle. The instrument is moved along the canal wall towards the apex. As soon as the guttapercha softens, the speed is increased to ○_{rev.} 4,000 rpm and the instrument is carefully guided further towards the apex. To this end, it is essential that the length of the canal be determined beforehand by means of a radiograph. This procedure should only take a few seconds, especially in the curved region of the root canal, because the rotation in the curved part of the canal increases the risk of fracture of the instrument. When the instrument is withdrawn, the softened guttapercha adheres to the instrument and can be removed with appropriate tools such as tweezers or by wiping the instrument with a cloth.

4. Further treatment

Final cleaning of the root canal (optinal), for example with a Hedstroem file.

5. Reprocessing

Those instrumnts that are supplied non-sterile have to be prepared prior to first use.

Manual pre-cleaning of the instruments:

Immerse the instrument in "Guttasolv" and leave to react for 2 minutes. Clean instrument with a cloth. Re-immers the instrument in "Guttasolv" and allow to react for 1 further minute. Clean thoroughly with a nylon brush 9873 and rinse under running water. Control the instrument visually to make sure that it is clean and undamaged. In case of residual contamination, repeat the above described cleaning steps until all visual contamination has been removed.
Reprocessing

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Instructions for use

Please read carefully and retain for future reference.

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6. Storage

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

7. Disposal

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

8. Safety and liability

See also "Instructions for use and safety recommendations" in the current Komet catalogue. The user is responsible for checking the products prior to use as to whether they are suited for the intended purpose. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user.

9. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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