Komet Dental  
 Gebr. Brasseler GmbH & Co. KG  
 Trophagener Weg 25  
 32657 Lemgo - Germany  
 Telefon+49 (0)5261 701- 0  
 Telefax+49 (0)5261 701- 289  
 info@kometdental.de  
 www.kometdental.de

**PathGlider**  
 © Gebr. Brasseler GmbH & Co. KG  
**3 10723 - Rev 2018-11**

Wurzelkanalaufbereitung, Root canal preparation,  
 Traitement endodontique, Preparación de canal  
 radicular, Strumentazione canalare

nur zum Einmalgebrauch, For single use only, Ne pas  
 réutiliser (usage unique), No reutilizar (un solo uso),  
 Esclusivamente monouso

Verpackungseinheit, Packing unit, Unité d'emballage,  
 Unidad de embalaje, Unità d'imballo

strahlensterilisiert (Sterile Instrumente sind  
 auf dem Verpackungsetikett entsprechend  
 gekennzeichnet), Sterilized using irradiation  
 (Sterile instruments are marked as such  
 on the label of the packaging), Stérilisation  
 par irradiation (Les instruments stériles  
 sont marqués comme tels sur l'étiquette  
 de l'emballage), Esterilización por radiación  
 (Los instrumentos estériles son marcados  
 como tales sobre la etiqueta del embalaje),  
 Sterilizzazione per mezzo di radiazione (Gli  
 strumenti sterili sono marcati come tali  
 sull'etichetta dell'imballaggio)

verwendbar bis (nur zutreffend bei sterilen  
 Instrumenten), Use-by date (only applies to sterile  
 instruments), A utiliser avant le (ne s'applique qu'aux  
 instruments stériles), Utilizar antes de (se aplica  
 solamente a instrumentos estériles), Data di scadenza  
 (riguarda soltanto strumenti sterili)

bei beschädigter Verpackung nicht verwenden  
 (nur zutreffend bei sterilen Instrumenten), Do not  
 use in case of damaged packaging (only applies to  
 sterile instruments), Ne pas utiliser si l'emballage  
 est endommagé (ne s'applique qu'aux instruments  
 stériles), No utilizar si el embalaje está dañado (se  
 aplica solamente a instrumentos estériles), Non  
 utilizzare se l'imballo è danneggiato (riguarda soltanto  
 strumenti sterili)

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**PathGlider files made of nickel-titanium with a taper of .03 for the mechanical glide path preparation of root canals**

**1. Indication**

The instruments are intended for the mechanical removal of diseased tissue and root filling materials within the course of a root canal treatment.

**2. Contraindication**

- The contra-indications are (e. g.):
- pronounced abrupt curves
- severely calcified root canals
- preparation of milk teeth

**3. Proper use**

- General information
- The instruments are designed for single use, i.e. on one patient only. Up to 5 root canals can be treated with each file.
- Make sure to observe the recommendations for the use of mechanical NiTi instruments and for the cleaning and disinfection of root canals.
- Check instruments for plastic deformations, e.g. untwisted blades, in the region of the working part during the course of the treatment, i.e. during intermediate cleaning. Discard any imperfect files.
- Determine the morphology of the canal by means of diagnostic radiographs.
- Prepare the canal access, for example with an opener (Komet OP10L19.204.030) or Gates burs (Komet G180 or G180A).
- Appropriate use of the PG03 files
- After having prepared the glide path with manual files (e.g. with "K" files) and determined the working length with an apex locating device, proceed as follows:
- Use in a torque-limited contra-angle or motor at a torque of 0,5 Nm and a speed of  $\varnothing_{rot}$  300 rpm (max. speed  $\varnothing_{max}$  500 rpm).
- Preparation of a glide path with manual files up to size 010, then use the file PG03 in size 015, followed by size 020 (optional)
- Preparation of a glide path with manual files up to size 015, then use the file PG03 in size 020
- Insert the files into the canal in rotary mode. The instruments are used in "pecking motion", i.e. the files are moved up and down the canal wall during a period of max. 5 - 8 seconds, according to the single-length principle.
- Clean the instruments periodically during the treatment, for example in the interim storage block 595. Rinse

the root canal thoroughly in order to flush out all debris.

**4. Further treatment**

Preparation of the root canal with e. g. rotary endodontic files like F360 or F6 SkyTaper. Please read carefully the instructions for use of file systems.

**5. Reprocessing**

The sterile packed files are not designed for reprocessing, i.e. cleaning/disinfection and sterilisation to enable their reuse. The instruments can only be used once (one patient only). Used instruments have to be discarded because a risk-free reuse of the products cannot be guaranteed. The reuse of these products poses a risk of infection and/or the safety of the products can no longer be guaranteed. Instruments in damaged packaging are no longer sterile and should not be used. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use and more detailed information, as amended, can be downloaded from our web site [www.kometdental.de](http://www.kometdental.de) or requested from the manufacturer Gebr. Brasseler.

**6. Storage**

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

**7. Disposal**

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

**8. Safety and liability**

See also "Instructions for use and safety recommendations" in the current Komet Dental catalogue. The user is responsible for checking the products prior to use as to whether they are suited for the intended purpose. The user is responsible for the appropriate use of the instruments. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to nonobservance of our recommendations for use or warnings as well as inadvertent misuse by the user.

**9. Procedure in case of serious adverse events**

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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