Komet Dental  
Gebr. Brasseler GmbH & Co. KG  
Trophagener Weg 25  
32657 Lemgo - Germany  
Telefon+49 (0)5261 701- 0  
Telefax+49 (0)5261 701- 289  
info@brasseler.de  
www.brasseler.de

## Opener

© Gebr. Brasseler GmbH & Co. KG  
**3 10796 - Rev 2018-11**

Wurzelkanalaufbereitung, Root canal preparation, Traitement endodontique, Preparación de canal radicular, Strumentazione canalare

Verpackungseinheit, Packing unit, Unité d'emballage, Unidad de embalaje, Unità d'imballo

strahlensterilisiert (Sterile Instrumente sind auf dem Verpackungsetikett entsprechend gekennzeichnet), Sterilized using irradiation (Sterile instruments are marked as such on the label of the packaging), Stérilisation par irradiation (Les instruments stériles sont marqués comme tels sur l'étiquette de l'emballage), Esterilización por radiación (Los instrumentos estériles son marcados como tales sobre la etiqueta del embalaje), Sterilizzazione per mezzo di radiazione (Gli strumenti sterili sono marcati come tali sull'etichetta dell'imballaggio)

verwendbar bis (nur zutreffend bei sterilen Instrumenten), Use-by date (only applies to sterile instruments), À utiliser avant le (ne s'applique qu'aux instruments stériles), Utilizar antes de (se aplica solamente a instrumentos estériles), Data di scadenza (riguarda soltanto strumenti sterili)

bei beschädigter Verpackung nicht verwenden (nur zutreffend bei sterilen Instrumenten), Do not use in case of damaged packaging (only applies to sterile instruments), Ne pas utiliser si l'emballage est endommagé (ne s'applique qu'aux instruments stériles), No utilizar si el embalaje está dañado (se aplica solamente a instrumentos estériles), Non utilizzare se l'imballo è danneggiato (riguarda soltanto strumenti sterili)

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### 1. Indication

The instruments are intended for the mechanical removal of diseased tissue and root filling materials within the course of a root canal treatment.

### 2. Contraindication

The contraindications are:

- pronounced abrupt curves
- severely calcified root canals
- preparation of milk teeth
- Do not use in the apical third of the root canal

### 3. Proper use

General information:

- Monitor and record carefully how many times each instrument has been used (no more than 8 should be prepared with one file).
- Be sure to observe the recommendations for the use of mechanical NiTi instruments and for the cleaning and disinfection of root canals.
- During the course of the treatment - i.e. during intermediate cleaning - check instruments for plastic deformations, e.g. untwisted blades in the region of the working part. Discard any imperfect files.

Appropriate use of the access reamer:

- The instruments OP10 & OP08 are used in torque-limited contra-angles at an optimum speed of 300 rpm (max. speed 500 rpm) and a torque of 2.8 Ncm. The instrument AK10 is used in a torque-limited contra-angle at an optimum speed of 500 rpm (max. torque 800 rpm) and a torque of 2.0 Ncm.
- The instruments OPR10 and OPR08 are used in commercially available, reciprocating power systems that allow full rotation of the instrument. Given that the blades of the instruments cut towards the left, most of the rotational movement must also be to the left. Failure to follow this instruction leads to jamming of the file and possibly to the fracture of the instrument because the efficiency of the file is insufficient when cutting to the right.
- Start up the file before inserting it into the root canal. The instruments are used in a “pecking motion”, i.e. performing brief upward and downward movements inside the canal. Preparation takes place along the coronal third of the root canal.
- Clean the instrument periodically during the treatment, for example in the interim storage block 595. Rinse the root canal thoroughly in order to flush out all debris.

### 4. Further treatment

The opened root canal can be prepared mechanically or manually after the preparation of a glide path, for example with the F360, F6 SkyTaper, Procodile or R6 ReziFlow system of NiTi files for the mechanical preparation of root canals provided by Komet.

### 5. Reprocessing

Sterile packed instruments can be used without previous processing, provided that the packaging is intact.

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## Instructions for use

Please read carefully and retain for future reference.

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## Opener made of NiTi, for the access enlargement as part of the preparation of root canals

## Opener made of NiTi, for the access enlargement as part of the preparation of root canals

## Opener made of NiTi, for the access enlargement as part of the preparation of root canals

## Opener made of NiTi, for the access enlargement as part of the preparation of root canals

Those instruments that are supplied non-sterile have to be prepared prior to first use. Komet Dental does not guarantee sterility of the instruments after each subsequent reprocessing cycle. The instruments are to be disinfected with anti-corrosive cleaning and disinfection agents for rotary instruments (e.g. with Komet DC1). For recommendations for use (immersion time, concentration, suitability) of cleaning and disinfection agents see instructions of the manufacturers of these agents. The instruments can be reprocessed in the thermo disinfectant, provided that the agent used is suitable for rotary instruments (mildly alkaline detergent). Cleaning and disinfecting agents must be rinsed off thoroughly with water and the instruments must be dried carefully (preferably with compressed air according to the recommendations of the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute). Do not store instruments for a longer period in wet or humid condition. Remove all traces of NaOCL and reduce the contact time to a minimum, i.e. a few minutes at most. Make sure that the instruments do not come in contact with each other during ultrasonic cleaning. Subject cleaned instruments to a visual examination. Damaged or blunt instruments must be rejected and their use discontinued.

Sterilization:

Sterilization of the instruments is carried out in the autoclave at 134°C. Observe instructions provided by the manufacturer. After sterilization, check instruments for defects on the surface (corrosion). Do not reuse corroded instruments. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious Disease Prevention of the Robert Koch Institute. Work instructions with regard to proper reprocessing of instruments according to DIN EN ISO 17664, the present instructions for use, as amended and more detailed information can be downloaded from our web site [www.kometdental.de](http://www.kometdental.de) or requested from the manufacturer Gebr. Brasseler.

### 6. Storage

Store packed instruments away from UV radiation and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals.

### 7. Disposal

The instruments have to be disposed of in tight, non-breakable and non-pierceable containers (protection from contamination).

### 8. Safety and liability

See also “Instructions for use and safety recommendations” in the current Komet Dental catalogue. The user is responsible for checking the products prior to use as to whether they are suited for the intended purpose. The user is responsible for the appropriate use of the instruments. In case of contributory negligence by the user, Gebr. Brasseler partially or totally declines liability for all resulting damages, particularly if these are due to nonobservance of our recommendations for use or warnings as well as inadvertent misuse by the user.

### 9. Procedure in case of serious adverse events

In case of serious adverse events, please proceed in line with the legal regulations in your country.

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